## *CardioMPO*<sup>™</sup> Calibrator Kit

Catalog Number: 9601

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# *CardioMPO*<sup>™</sup> Calibrator Kit

## **INTENDED USE**

The PrognostiX *CardioMPO*<sup>™</sup> Calibrator Kit is intended for use with the *CardioMPO*<sup>™</sup> Reagent Kit to establish a calibration curve that is used to determine MPO concentration.

## **SUMMARY AND EXPLANATION**

The use of Calibrators is designed to assure accurate results over the linear range of the test. The *CardioMPO*<sup>™</sup> Calibrator Kit contains five levels of MPO provided ready-to-use in frozen liquid form for extended shelf life.

## **REAGENTS AND MATERIALS**

### **Materials supplied with the kit:**

#### ***CardioMPO*<sup>™</sup> Calibrators, 5 vials, MPO levels A-E, 1.1 mL each**

Five concentrations of human MPO in a phosphate-buffered matrix containing proteins, detergents, and stabilizers. Refer to individual Calibrator vial labels for actual concentrations of Calibrators.

### **Materials required but not supplied:**

*CardioMPO*<sup>™</sup> Enzyme Immunoassay Reagent Kit                      Cat. Number 7601

## **WARNINGS AND PRECAUTIONS**

For *in vitro* diagnostic use.

For professional use only.

## **REAGENT STORAGE**

Unopened vials must be stored frozen at -15 to -20°C. The Calibrators are provided ready to use.

## **STABILITY**

### **Unopened frozen vials:**

See the Expiration Date on the Calibrator label.

### **Thawed vials:**

The assigned values are stable for 14 days when recapped and stored at 2-8°C. Thawed vials that are in use or stored for longer than 1 hour should be recapped tightly to prevent evaporation.

## **INDICATIONS OF INSTABILITY OR DETERIORATION OF CALIBRATORS**

Store *CardioMPO*<sup>™</sup> Calibrators at the labeled temperature and do not use past their expiration dates.

Do not use Calibrators that have indications of discoloration, cloudiness or precipitation. Do not use Calibrators that show signs of leakage.

### **ASSAY PROCEDURE**

*CardioMPO*<sup>™</sup> Calibrators are added directly to the *CardioMPO*<sup>™</sup> Reagent Kit microtiter plate and run in accordance with the procedure found in the package insert for the *CardioMPO*<sup>™</sup> Reagent Kit.

Before use, thaw at room temperature (20 – 26°C) for a minimum of 30 minutes. Thoroughly mix the contents of the vial while avoiding foaming.

### **EXPECTED VALUES**

Calibrator values are assigned for each lot. Refer to individual Calibrator vial labels for actual concentrations of Calibrators.

**WARRANTY**

This product is warranted to perform as described in its labeling and in Prognostix, Inc.'s literature for this product when used in strict conformity with published directions regarding the use of this product. Prognostix, Inc. makes no other warranty, express or implied, and **DISCLAIMS ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. UNDER NO CIRCUMSTANCES WHATSOEVER SHALL PROGNOSTIX, INC. BE LIABLE FOR ANY INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.**

Manufactured for:  
PrognostiX, Inc.

10265 Carnegie Avenue, Cleveland, Ohio 44106

Tel: (866) 358-9828, (216) 445-7469 (PGNX)

Fax: (216) 444-5335

info@prognostix.com

www.prognostix.com

Authorized Representative:

Emergo Europe

Molenstraat 15

2513 BH, The Hague

The Netherlands

Tel: +31 (0)70 345 8570

Fax: +31 (0)70 346 7299

**EC**

**REP**

This product is covered by U.S. Patent Application 20020164662.  
*CardioMPO™* is a pending trademark of PrognostiX, Inc.

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*CardioMPO™* è un marchio in corso di registrazione di Prognostix, Inc.